



# Cardio Partners

Capable. Prepared. Responsive.

## AED Inspection Form V4.0

Today's Date: \_\_\_\_\_

Company Name: <b>Bank of America</b>	Location ID	Address/ City/ State/Zip:  <i>Location ID and address confirmed by an Associate? Y N</i>
Inspected By:	Location of AED in building: Be Specific	
Managers Name:	AED Model <b>Cardiac Science G5</b> <i>If G3, text 614.760.5067 immediately</i>	Serial Number: <b>D000000</b>

Use the following maintenance checklist when you check **EACH** unit.

Check the following	PASS	FAIL	Comments
Is the Readiness Indicator Green?	<input type="checkbox"/>	<input type="checkbox"/>	
Is the unit clean, undamaged, and free of excessive wear?	<input type="checkbox"/>	<input type="checkbox"/>	
Open AED lid. Are lights on battery indicator bar (bottom left) green? Fail if light is red.	<input type="checkbox"/>	<input type="checkbox"/>	How many lights are green on the battery indicator bar?
Is the Audio working properly and text aligned with audio?	<input type="checkbox"/>	<input type="checkbox"/>	
Unplug pads. Wait for indicator to turn red then plug back in. Wait 60 seconds. Does the indicator return to green?	<input type="checkbox"/>	<input type="checkbox"/>	
Does spanish button (right button) illuminate when clicked and change language?	<input type="checkbox"/>	<input type="checkbox"/>	
Close lid making sure pad exp. date is visible through window of the lid.	<input type="checkbox"/>	<input type="checkbox"/>	Pad Exp. Date:
Place AED in carry case and in center of cabinet facing outward. Indicator and exp. should be visible through window.	<input type="checkbox"/>	<input type="checkbox"/>	
Check for adequate supplies (mask, gloves, extra batteries)	<input type="checkbox"/>	<input type="checkbox"/>	
Alarm on AED Box operates correctly and batteries were replaced.	<input type="checkbox"/>	<input type="checkbox"/>	

**Comments:**


**UPLOAD FORM VIA THE CARDIO PARTNERS PORTAL  
ANY QUESTIONS OR CONCERNS PLEASE CALL OR TEXT 614.760.5067.**