



Today's Date: _____

Company Name:	Store #	Address/ City/ State:
Inspected By:		Location of Inspection (Building): Be Specific
Managers Name & CELL #:	AED Model	Serial Number:

Use the following maintenance checklist when you check **EACH** unit.

***** WIFI SETUP MUST BE CHECKED, MUST BRING A PC (NOT MAC) WITH MICRO USB CABLE TO CONNECT TO THE DEVICE. CALL CHRISTIAN IF UNIT IS NOT REPORTING READY ON WIFI. 24x7 LINE 214-501-5104. DO NOT LEAVE UNTIL THE WIFI IS CONFIRMED CONNECTED *****

Check the following	PASS	FAIL	Comments
Is the Readiness Indicator Green or ready to use?	<input type="checkbox"/>	<input type="checkbox"/>	
<u>Is units WIFI setup (PC Required) and reporting Online Status Ready?</u>	<input type="checkbox"/>	<input type="checkbox"/>	
Is the unit clean, undamaged, and free of excessive wear / cracks?	<input type="checkbox"/>	<input type="checkbox"/>	
Verify that electrodes are within their expiration date.	<input type="checkbox"/>	<input type="checkbox"/>	
Verify electrodes are connected to the unit and sealed in their package. Replace if expired.	<input type="checkbox"/>	<input type="checkbox"/>	Exp. Date:
Turn the unit on and off and verify the unit ready for use.	<input type="checkbox"/>	<input type="checkbox"/>	
Batteries within expiration date. Replace if expired.	<input type="checkbox"/>	<input type="checkbox"/>	Exp. Date:
Check for adequate supplies (mask, gloves, extra batteries)	<input type="checkbox"/>	<input type="checkbox"/>	
Alarm on AED Cabinet operates correctly.	<input type="checkbox"/>	<input type="checkbox"/>	
Comments:			

UPLOAD FORM VIA THE CARDIO PARTNERS PORTAL
ANY WIFI ISSUES, QUESTIONS OR CONCERNS PLEASE
CALL 866-599-2337 ext. 4. or Christian Hernandez @ 214-501-5104