



Cardio Partners

Capable. Prepared. Responsive.

AED Inspection Form V3.0

Today's Date: _____

Company Name: The Home Depot	Store #	Address/ City/ State/Zip: <i>Store # and address confirmed by an Associate? Y N</i>
Inspected By:	Location of AED in building: Be Specific	
Managers Name:	AED Model Cardiac Science G5	Serial Number: D000000

Use the following maintenance checklist when you check **EACH** unit.

Check the following	PASS	FAIL	Comments
Is the Readiness Indicator Green?	<input type="checkbox"/>	<input type="checkbox"/>	
Is the unit clean, undamaged, and free of excessive wear?	<input type="checkbox"/>	<input type="checkbox"/>	
Open AED lid. Are lights on battery indicator bar (bottom left) green? Fail if light is red.	<input type="checkbox"/>	<input type="checkbox"/>	How many lights are green on the battery indicator bar?
Is the Audio working properly and text aligned with audio?	<input type="checkbox"/>	<input type="checkbox"/>	
Unplug pads. Wait for indicator to turn red then plug back in. Wait 60 seconds. Does the indicator return to green?	<input type="checkbox"/>	<input type="checkbox"/>	
Does spanish button (right button) illuminate when clicked and change language?	<input type="checkbox"/>	<input type="checkbox"/>	
Close lid making sure pad exp. date is visible through window of the lid.	<input type="checkbox"/>	<input type="checkbox"/>	Pad Exp. Date:
Place AED in carry case and in center of cabinet facing outward. Indicator and exp. should be visible through window.	<input type="checkbox"/>	<input type="checkbox"/>	
Check for adequate supplies (mask, gloves, extra batteries)	<input type="checkbox"/>	<input type="checkbox"/>	
Alarm on AED Box operates correctly and batteries were replaced.	<input type="checkbox"/>	<input type="checkbox"/>	

Comments:

**UPLOAD FORM VIA THE CARDIO PARTNERS PORTAL
ANY QUESTIONS OR CONCERNS PLEASE CALL 866-599-2337 ext 4.**