AED Inspections

Today's Date:

for Cardiac Science AEDs



Capable. Prepared. Responsive.

Company Name:	Location ID	Address/ City/ State/Zip:		
		Location	n ID and address confirmed by an Associate? Y N	
nspected By:		Location of AED in building: Be Specific		
Managers Name:	AED Model		Serial Number:	
	Cardiac Science G5		D000000	

Use the following maintenance checklist for **EACH** unit.

IF JOB INCLUDES ACCESSORY REPLACEMENT, PLEASE REPLACE BEFORE PROCEEDING.					
ALL STEPS ARE REQUIRED!	PASS	FAIL	Comments		
Is the Readiness Indicator Green?					
Are pads current? If not, please replace.			Pad Exp. Date:		
Open lid. Once it starts to speak, click and hold both gray buttons for 5-10 seconds until it states "diagnostic mode"			Toggle to show battery info screen. What is the capacity %:		
Is the Audio working properly and text aligned with audio?					
Unplug pads. Wait for indicator to turn red then plug back in. Wait 60 seconds. Does the indicator return to green?					
Does spanish button (right button) illuminate when clicked and change language?					
Is the unit clean, undamaged, and free of excessive wear?					
Place AED in center of cabinet facing outward with indicator and exp. visible through window.					
Check for adequate supplies in ready kit (mask, gloves, etc.)					
Alarm on AED Box operates correctly.			If fail, let site contact know to replace batteries.		
Did you replace any of the following today? (circle):			PADS BATTERY		
Comments:					